


# COMBINED TABLE

(Model: 700X-2)

## USER MANUAL

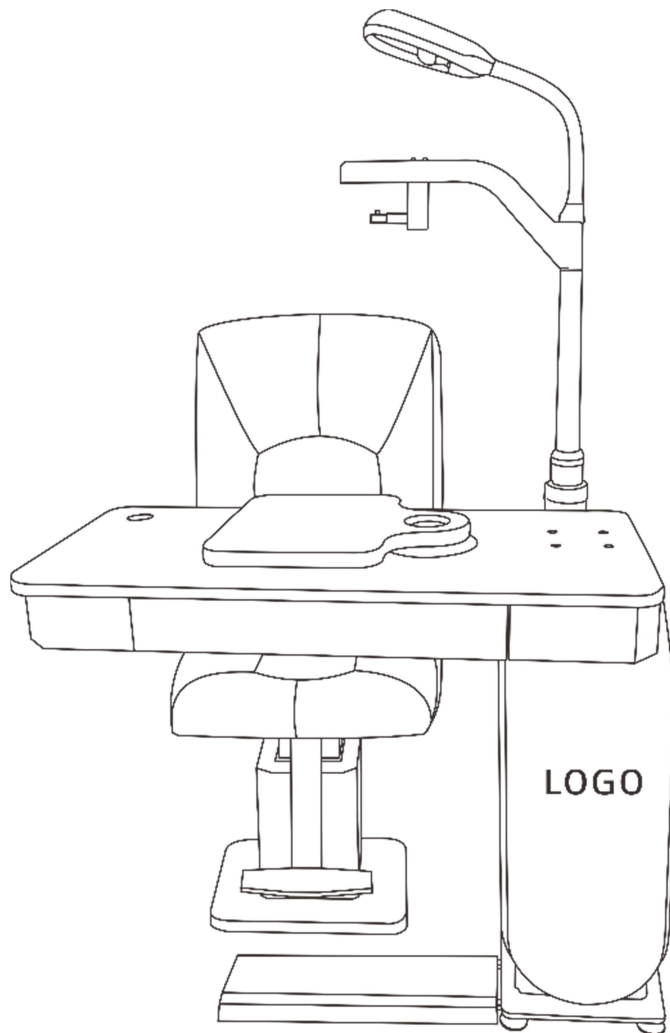

----1. Specification

Big table-top size	1045×485mm	The height of the big table-top	860mm
Small table-top size	400×300mm	The height of the small table-top	900mm
Drawer inner size	540×390×57mm	The height of the phoroptor stand	1410-1720mm
Arm up and down range	310mm	Lighting parameters	24V, 6W
Arm rotation angle	90°	Input power	100W
Table-top rotation angle	90°	Input voltage	220V, 50Hz or 110V, 60Hz


----2. Installation (Main Body)

1. Unpack the table-top box, take out the accessories package, here is the list:

inner-hexagonal screws M8 *5 & M10*5	M8 washer*5	Cable of chair*1
Silicone blocking for the table-top*5	M5 truss head screw*5	Cross screwdriver*1
M8 spring washer*5	Main machine power cable*1	8A fuse*2(spare)
Allen key*6 (M2.5/3/4/5/6/8)		




2. Unpack the carton of main machine, take down the side cover (magnetic adsorption, pull it out from the bottom first).

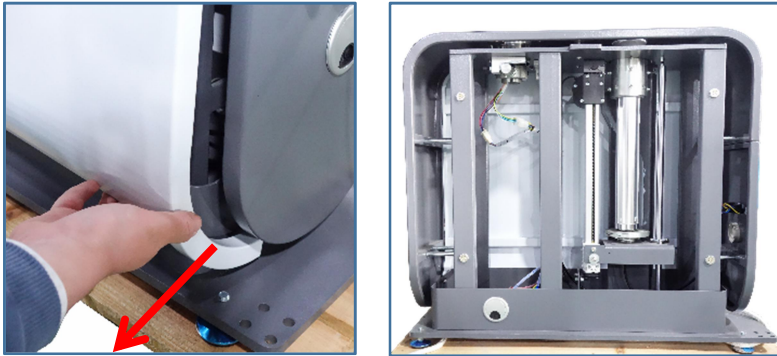

3. Remove the two screws fixed on the tray (as shown in the figure below), lift the main machine and place it on the ground, then adjust the four foot pads for stable placement.



4. Take out the pedal support, assemble the inner hexagon screws (M8\*4pcs) in the corresponding holes, adjust the height of pedal pad if the ground is not flat.

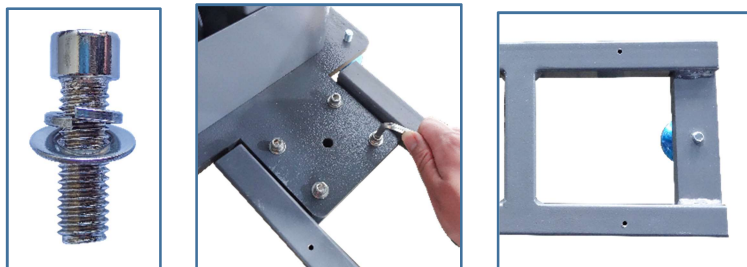


5. Take out the pedal, and screw 4pcs of M5 truss head screws.



6. Take out the phoroceptor arm, connect the wires between the main machine and the arm, insert the wire into the pipe and then put the arm in.





7. Adjust the direction of the phoroceptor arm to be perpendicular to the main machine, use an Allen key to lock the screw which are inside the arm.

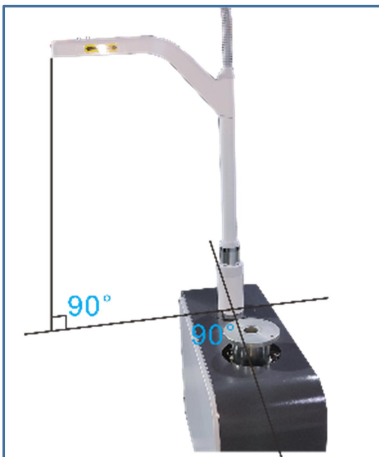


8. Put the big table-top above the main machine, and insert the two cables into the bearing hole (as shown in the figure below).





9. After putting on the table-top, screw 4pcs of M10 inner-hexagonal screws and cover 4pcs of silicone blockings.

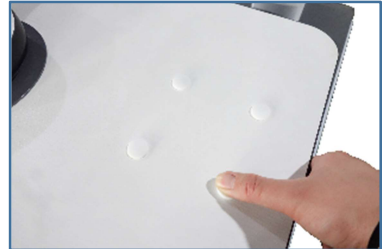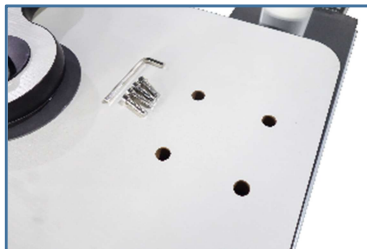

10. Connect two sets of plugs in the main machine box, 4pin match with 4pin, 9pin match with 9pin.

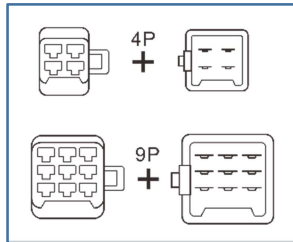

11. Put the side cover back, first hang the top on the buckle, and then slowly push the bottom in along the magnetic attraction.

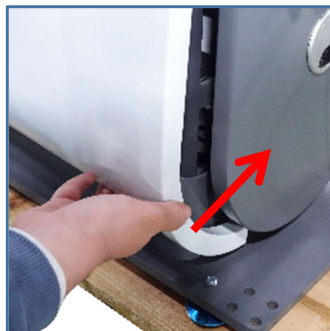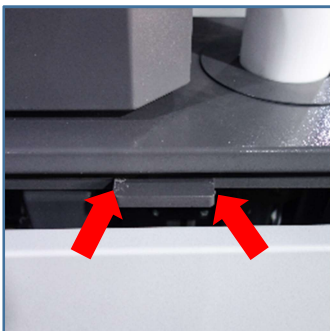

### 3. Installation (Motorized Chair)

1. Unpack the motorized chair carton, take out the accessories package, the list is as follows.

M8 inner-hexagonal screws *8	M6 inner-hexagonal screws *6
Silver blocking *6	Black blocking *2



2. Loosen the screws which fix the chair on the wooden box, take out the base.



3. Take out the seat cushion, press down the leather layer, the nut inside will be exposed (as shown in the picture), align the armrest on this side with the screw holes, and screw in the M8 inner-hexagonal screws.



4. Take out the chair back, align the nut and screw in the M8 inner-hexagonal screws, repeat step 3 to complete the installation of the other armrest, and finally press the blocking into the screw hole with the corresponding colors.



5. Tips, you could power on and rise the base into highest level, it will make the installation in the next step more easily.



6. Put the well assembled chair body on the chair base, align the holes and screw in 6 pcs of M6 inner-hexagonal screws from the bottom, then all the installations are finished.

